





Instructions for Use

ASPIRE™ M2

		3
EN	Instructions for Use	4
DE	Gebrauchsanweisung	7
FR	Notice d'utilisation	11
ES	Instrucciones para el uso	15
IT	Istruzioni per l'uso	19
NO	Bruksanvisning	23
DA	Brugsanvisning	26
SV	Bruksanvisning	29
EL	Οδηγίες Χρήσης	32
FI	Käyttöohjeet	36
NL	Gebruiksaanwijzing	39
PT	Instruções de Utilização	43
PL	Instrukcja użytkowania	47
TR	Kullanım Talimatları	51
RU	Инструкция по использованию	54
JA	取扱説明書	58
ZH	中文说明书	61
KO	사용 설명서	64

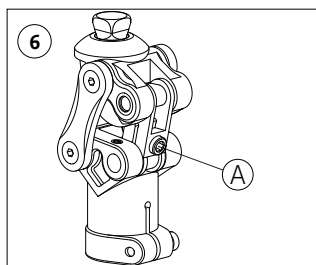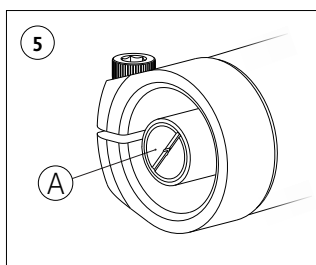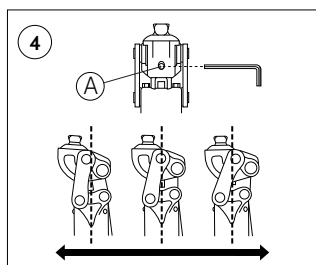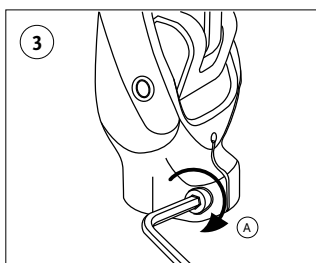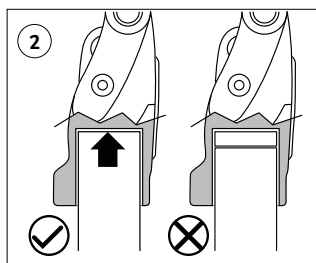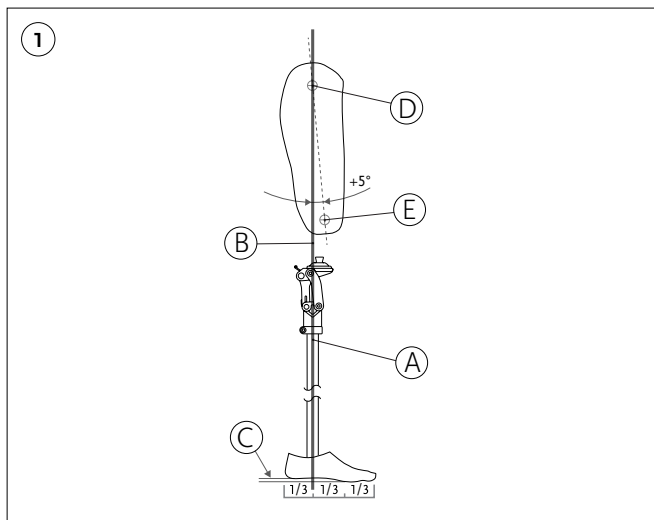DESCRIPTION

The device is a four-bar polycentric mechanical knee. It is designed to facilitate single cadence ambulation on level ground and small slopes. Extension of the knee is aided by a spring, which is compressed during knee flexion.

The device has a Distal Tube Connector and a Proximal Integrated Male Pyramid Adapter.

INTENDED USE

The device is intended as a part of a prosthetic system that replaces knee function of a missing lower limb.

Suitability of the device for the prosthesis and the patient must be evaluated by a healthcare professional.

The device must be fitted and adjusted by a healthcare professional.

Indications For Use and Target Patient Population

- Lower limb loss, amputation, or deficiency
- No known contraindications

The device is for low impact use, e.g., gentle walking.

The weight limit for the device is 136kg.

GENERAL SAFETY INSTRUCTIONS

Warning: Use of a lower limb prosthetic device carries an inherent risk of falling which may lead to injury.

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

Warning: If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

Warning: Avoid placing hands or fingers near moving joints.

Caution: Do not adjust other screws than described in these instructions.

The device is for single patient use.

ALIGNMENT INSTRUCTIONS

Bench Alignment (Fig. 1)

Alignment Goal

Alignment reference line (B) should:

- pass through midpoint of socket at the ischial tuberosity level (D)
- pass through midtube (A)
- fall at the 1/3 mark on the inside of the foot cover.

Note: Prioritize knee alignment over foot alignment if there is a mismatch.

Alignment Instructions

1. Position the foot so that the alignment reference line (B) falls at the 1/3 mark on the inside of the foot cover (with the foot cover and shoe on). Consider the external rotation of the foot.

2. Use the applicable adapters to connect the knee to the foot and establish the correct knee center height.
3. Position the knee so that the alignment reference line passes through midtube (A)
4. On the lateral side of the socket, make a first mark at the midpoint of the socket at the ischial tuberosity level (D). Make a second mark at the midpoint of the socket distally (E). Draw a line through both marks.
5. Position the socket so the alignment reference line (B) goes through first mark at the midpoint of the socket at the ischial tuberosity level (D).
6. Adjust the socket flexion to 5° in addition to the existing position (i. e., hip flexion contracture) and set the height of the full prosthesis.
7. Use the applicable adapters to connect the knee to the socket.

Warning: After adjustments, all screws must be secured with a medium strength threadlocker and tightened with the correct torque.

Caution: Adapters used on the distal connection must be cut straight and inserted down to the end stop of the device's tube receiver. No spacer should be used (**Fig. 2**).

Tighten the screws with the following torque:

- Tube Clamp Screw (**Fig. 3**): 16 Nm

Static Alignment

- Make sure the patient stands with equal weight on both legs.
- Check for correct prosthesis length.
- Check internal / external rotation.
- Check for correct load on toe and heel.

Dynamic Alignment

Ensure the patient is familiar with the functioning of the device.

Ensure the patient can activate and deactivate the geometric lock.

Adjustment of stance phase stability (Fig. 4)

Stance phase stability can be adjusted by altering the 5 mm Geometry Screw (A) to change axial arrangement according to the following:

- To increase stance phase stability: turn the 5 mm Geometry Screw to the left
- To allow for more dynamic gait: turn the 5 mm Geometry Screw to the right

Always check the result right after each adjustment step.

Swing control

Adjusting the Extension Assist Spring (**Fig. 5**):

Turn the Extension Spring Adjustment Screw (A) to the:

- right to increase the action of the Extension Spring.
- left to decrease the action of the Extension Spring.

Adjusting axle friction (Fig. 6):

Turn the Friction Screw (A) to the:

- right to increase friction on the axle.
- left to decrease friction on the axle.

USAGE

Cleaning and care

Wipe the device with a soft cloth. Do not use solvents.

Environmental Conditions

The device should not come in contact with fresh water, salt water or chlorinated water.

Caution: Device should not be used in dusty environment. Exposure to sand, talcum, or similar should be avoided.

The device can be used in temperatures between -10°C to 40°C.

MAINTENANCE

The device and the overall prosthesis should be examined by a healthcare professional. Interval should be determined based on patient activity.

Recommended interval is every 6 months.

Check for damage, excessive wear and dirt.

Wipe the device with a soft cloth moistened with small amount of general purpose oil or sewing machine oil.

Caution: Do not use compressed air to clean device. Air forces pollutants into bearings and may cause malfunctions and wear.

Caution: Do not use talcum powder for lubrication.

REPORT OF SERIOUS INCIDENT

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur prosthetic devices are designed and verified to be safe and compatible in combination with each other and custom-made prosthetic sockets with Össur adapters, and when used in accordance with their intended use.

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

Compliance

This device has been tested according to ISO 10328 standard to three million load cycles.

Depending on patient activity, this may correspond to 3-5 years of use.

ISO 10328 - P6 - 136 kg *)

*) Body mass limit not to be exceeded!

For specific conditions and limitations of use see manufacturer's written instructions on intended use!

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