## ENGLISH

### MD Medical Device

**DESCRIPTION**  
The device is a Ratchet Lock, a lock system to be used in conjunction with locking liners.

**Components (Fig. 1)**  
1. Attachment Pin  
2. Funnel  
3. Ratchet Lock  
4. Release Unit  
5. Release Key  
6. Fabrication Ring  
7. Cross Driver

**INTENDED USE**  
The device is intended to connect and release a prosthetic system that replaces a missing lower limb.

Suitability of the device for the prosthesis and the patient must be evaluated by a healthcare professional.

The device must be fitted and adjusted by a healthcare professional.

#### Indications For Use and Target Patient Population

- Lower limb amputation and/or congenital deficiency
- No known contraindications

The device is for high impact use, e.g., walking and occasional running.

#### GENERAL SAFETY INSTRUCTIONS

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

**Warning:** If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

The device is for single patient use.

#### ASSEMBLY INSTRUCTIONS

**Note:** Use with L-780000 Fabrication Tooling Kit (reusable)

#### Installation:

1. Place Ratchet Lock into the bottom of the socket with release unit hole incorrect position.
2. Shorten Release Unit to the required length and screw it firmly and completely into the hole on socket and Ratchet Lock.
- Note:** L-792016 Release Key makes installation and removal of Release Unit easier and is ideal when using a prosthesis with an exoskeletal design.
3. Screw Funnel into threads above Ratchet

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**Attachment Pin**  
Apply Locite 410/411 onto the threads on the Attachment Pin and screw it into the Locking Liner. Torque the Attachment Pin to 4 Nm.  
**Note:** Use Attachment Pins for Icelock 700 series only.

**Caution:** Ensure that the Pin is fully engaged. If temporary swelling prevents full engagement, a minimum of four teeth shall enter the lock for safety reasons.

#### USAGE

**Cleaning and care**  
Clean with a damp cloth and a mild soap. Dry with a cloth after cleaning.

#### Environmental Conditions

The device is Weatherproof. A Weatherproof device can be used in a wet or humid environment and can tolerate being splashed by fresh water (e.g., rain), no submersion is allowed.

No contact with salt water or chlorinated water is allowed.

Dry with a cloth after contact with fresh water or humidity.

Clean with fresh water in case of accidental exposure to other liquids, chemicals, sand, dust, or dirt and dry with a cloth.

#### Maintenance

The device and the overall prosthesis should be examined by a healthcare professional.

**Warning:** If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

The device is for single patient use.

#### ASSEMBLY INSTRUCTIONS

**Note:** Use with L-780000 Fabrication Tooling Kit (reusable)

#### Installation:

1. Place Ratchet Lock into the bottom of the socket with release unit hole incorrect position.
2. Shorten Release Unit to the required length and screw it firmly and completely into the hole on socket and Ratchet Lock.
- Note:** L-792016 Release Key makes installation and removal of Release Unit easier and is ideal when using a prosthesis with an exoskeletal design.
3. Screw Funnel into threads above Ratchet

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