3. Screw the Bottom Attachment to the Guide with the Screw. Make sure that the exit hole, with the Dummy for the Lanyard Cord, is correctly positioned to ensure that the Lanyard operates with minimum friction.
4. Vacuum-form the thermoplastic, ensuring that the plastic shapes accurately to the Bottom Attachment.
5. After manufacturing the outer socket, trim around the Screw and the exit hole and remove the Screw and the Dummy.
6. Assemble the socket and drill a 5mm hole through the outer socket - opposite and slightly higher than the exit hole.
7. Thread the Lanyard Cord through the Bottom Attachment and out through the exit hole.
8. Find the correct positioning for the Cleat; drill two 3mm holes and rivet it to the socket.
9. Trim the Cord Cover to the desired length, insert it into the exit hole and glue it to the socket in the correct location, using Locite 410/411.

Cleat Positioning (Fig. 3)

10. Released
11. Locked

Note: The Pull Handle can be used for getting a more secure grip on the Lanyard Cord when the socket is donned.
Caution: Tie a knot in the Lanyard Cord to stop it from inadvertently being pulled into the channel.

USAGE

Cleaning and care

Clean with a damp cloth and a mild soap. Dry with a cloth after cleaning.

Environmental Conditions

The device is Weatherproof.
A weatherproof device can be used in a wet or humid environment and can tolerate being splashed by fresh water (e.g., rain), no submersion is allowed.
No contact with salt water or chlorinated water is allowed.

Dry with a cloth after contact with fresh water or humidity.
Clean with fresh water in case of accidental exposure to other liquids, chemicals, sand, dust, or dirt and dry with a cloth.

Maintenance

The device and the overall prosthesis should be examined by a healthcare professional. Interval should be determined based on patient activity.

Report of Serious Incident

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Ossur does not assume liability for the following:
• Device not maintained as instructed by the instructions for use.
• Device assembled with components from other manufacturers.
• Device used outside of recommended use condition, application, or environment.

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