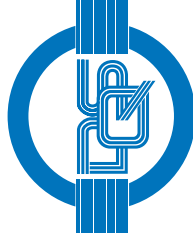
Medical Line

Medical Line

DIN EN ISO 9001:2000
DIN EN ISO 13485:2001

AchilloTrain® Pro

Aktivbandage
Active support
Orthèse active
Vendaje activo

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ENGLISH

Dear client,

Thank you for choosing a product from Bauerfeind Orthopedics. AchilloTrain Pro is a high-quality medical product.

Please read through these instructions for use carefully. If you have any questions please contact your doctor, your local retailer or our Technical Service department.

AchilloTrain Pro, the active Achilles tendon support is anatomically knitted which ensures an excellent fit. The massage effect of the shaped viscoelastic insert around the Achilles tendon area promotes lymph drainage and circulation and helps reduce swelling and bruising. The stretchable knitted section around the shin also facilitates donning of the support.

AchilloTrain Pro must be fitted for the first use by a trained specialist, otherwise we cannot accept any liability for the product.

Indications

- Painful irritation of the Achilles tendon (achillodynia) and adjacent tissues
- After surgery, e.g. tears in Achilles tendon

Side effects

Side effects involving the body as a whole have not been reported to date. Correct fitting is assumed. Any supports and orthoses¹ applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves.

Contra-indications

Hypersensitive reactions harmful to health have not been reported to date.

In the following conditions such aids should only be fitted and worn after consultation with your doctor:

1. Skin disorders/injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
2. Impaired sensation and circulatory disorders of the foot, e.g. Diabetes mellitus.
3. Impaired lymph drainage – including soft tissue swellings of uncertain origin located remotely from the fitted aid.

¹ Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

Instructions for use

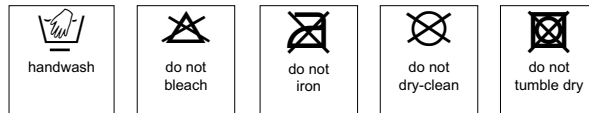
Fitting AchilloTrain Pro

1. Grasp the top edge of the support and pull it over the foot until the heel is located in the heel-wedge section (Figures 1 and 2).
2. Ensure that the Achilles tendon is securely held by the shaped insert.
3. Adjust the fit of the support as necessary.

Important Information

- AchilloTrain Pro is a prescribed product that should be worn under a physician's guidance. AchilloTrain Pro should only be worn in accordance with the instructions contained in these usage directions and for the listed areas of application.
- The product is intended to be fitted only once and to one patient.
- No liability is accepted for improper use of the product.
- Changes must not be made for the product. Non compliance can reduce the performance of the product and thus no liability will be accepted.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your doctor without delay.
- A combination with other products (e.g. compression stockings) must be discussed beforehand with your doctor.
- The product's unique action is produced primarily during physical activity. The product should be removed during prolonged rest breaks (e.g. sleeping, continuous sitting).
- Avoid the use of ointments, lotions or any substances including grease or acids.
- Guarantee subject to statutory regulations.
- The product can be disposed of without any restrictions.

Washing instructions and product care

AchilloTrain Pro can be washed separately by hand with a mild detergent at low temperatures. Do not dry in direct heat (e.g. heater, sunlight, etc.).

This product has been examined by our own quality control system. However, should you have any complaint, please contact your retailer or our customer service department.

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