Rev. 1 - 04/11_108824

| Material |
|-----------------------|
| Polyamid (PA) |
| Stäbe (Al Mg3) |
| Polyurethan (PUR) |
| Polyethylen (PE) |
| Polyoxymethylen (POM) |
| Polyester (PES) |

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EN Dear customer,

Thank you for placing your trust in a Bauerfeind product. The ManuRhizoLoc you have purchased is a high quality medical product.

Please read through all of these instructions for use carefully, because they contain important information about the use, indications, contraindications, application, care and disposal of the ManuRhizoLoc. Please keep these instructions safe, as you may want to read them again later. If you have any questions, please contact your medical advisor or medical retailer.

Intended use

The ManuRhizoLoc is an anatomically contoured stabilizing orthosis for the lower arm, wrist and thumb. The ManuRhizoLoc stabilizes and, if necessary, immobilizes the wrist, preventing undesired movements over the whole wrist area. This is made possible by two lateral stay and a central stay. Undesired movements in the area of the thumb joint are also minimized. A partial immobilization of the thumb basal joint and thumb carpometacarpal joint is achieved, whereby the mobility of the end joint is not limited.

With its stabilizing effect, the ManuRhizoLoc helps prevent undesired movements and relieve pain.

Indications

- pre-operative
- irritation in the wrist or in the thumb basal joint and thumb carpometacarpal joint (post-traumatic and post-operative)
- severe carpometacarpal joint arthrosis
- combination injury in the wrist and thumb carpometacarpal joint
- carpal tunnel syndrome

Application risks

Caution*

ManuRhizoLoc must only be worn in accordance with the specifications contained in these instructions and used for the areas of application listed. To guarantee the best possible fit of the ManuRhizoLoc, it is essential to check whether the product is adequately shaped and whether it needs to be anatomically reshaped by a specialist¹.

The sewn-in label of the ManuRhizoLoc, which contains information on the product name, size, manufacturer, washing instructions and CE certification, is located in the insertion channel of the central stay on the inside of the lower arm splint.

¹ A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of orthoses which are relevant to you.

* Note on the risk of personal injury (risk of injury, harm to health and accidents) or damage to property (damage to the product).

Use in combination with other products, e. g. as part of compression treatment (arm stockings), must be discussed beforehand with your physician.

Should you notice any unusual changes (e.g. increased symptoms), please contact your physician without delay.

No product liability is accepted in the event of improper use.

Avoid the use of ointments, lotions or any substances including grease or acids.

Do not make any changes to the product. Failure to comply with this requirement may adversely affect product performance, thereby voiding any product liability.

Side effects involving the body as a whole have not been reported to date. Correct fitting is assumed. Do not put the ManuRhizoLoc on too tightly as this could lead to local pressure symptoms. In rare cases, constriction of blood vessels and nerves may occur. If this happens, loosen the straps of the ManuRhizoLoc and, if necessary, check the size of your ManuRhizoLoc.

Contraindications

Hypersensitivity reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your physician:

- skin disorders/injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up
- impaired sensation and circulatory disorders of the arm/hand (e. g. diabetes mellitus)
- impaired lymph drainage, including soft tissue swellings of uncertain origin located away from the fitted support

Application instructions

Putting on the ManuRhizoLoc

A - lower arm splint (lower arm section), B - thumb splint (thumb section), C - Velcro tab for controlling the mobility of the thumb

Together with your physician, you have ascertained the extent to which the movement of your thumb should be restricted. You should only change this following consultation with your physician.

- 1 Start by loosening the three Velcro straps on the lower arm splint.
- 2 Loosen the upper strap on the thumb splint without pulling it out of the slot.
- 3 Put the ManuRhizoLoc onto the affected arm. The straps are thus located on the top of the lower arm and on the back of the hand.
- 4 Fasten the straps on the lower arm splint in the following sequence: central strap, lower arm strap, and strap over the back of the hand.
- 5 The fasten the upper thumb strap and secure it toward the wrist.

Taking off the ManuRhizoLoc

Loosen the upper thumb strap. Then loosen the straps on the lower arm splint. Take off the ManuRhizoLoc.

Cleaning instructions

Note: Never expose the ManuRhizoLoc to direct heat (e.g. heaters, sunlight, in your car, etc.)! This may damage the material, which can impair the effectiveness of the ManuRhizoLoc.

Fasten all Velcro straps before washing to avoid damage. Wash the ManuRhizoLoc separately by hand, using a mild detergent.

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Assembly and fitting instructions

The ManuRhizoLoc is supplied in an anatomically contoured state. The orthosis is available for both the right and left hand.

10 Technical specifications / parameters, accessories

The ManuRhizoLoc is a stabilizing orthosis for the lower arm, wrist and thumb. It consists of a lower arm splint and a thumb splint. The thumb splint and the straps on the lower arm splint are detachable. There are no spare parts available.

Notes on reuse

The ManuRhizoLoc is intended for your treatment only. It has been adapted specifically for you. For this reason, do not pass the ManuRhizoLoc on to other people.

Warranty

Warranty is provided in accordance with the statutory provisions as well as the terms of supply and contractual terms of Bauerfeind AG (AGB), as long as all the points mentioned above are observed.

If you have any complaints, please contact your medical retailer only. We would point out that only clean goods can be dealt with.

Disposal

You can dispose of the product in accordance with national legal requirements.

Version: February 2011

Specialists

General notes

- Check the optimal and individual fit of the ManuRhizoLoc after the first fitting. Make corrections if necessary.
- Practice the correct procedure for putting on the orthosis with the patient.
- The product must not be changed in any improper way. Failure to comply with this requirement may adversely affect the medical and therapeutic effectiveness of the product, thereby voiding any product liability.
- The ManuRhizoLoc is intended for the treatment of one patient only.

The aluminum stays are anatomically contoured, but they can still be individually adjusted to suit the patient's anatomy.

Shaping the stays:

I Carefully remove the stays from the insertion pockets and shape them to the anatomy of the patient. **II** Reinsert the three stays into the pockets provided.

Adjust the thumb section to the length of the patient's thumb:

III Loosen the lower Velcro section of the thumb splint. Adjust this to the length of the patient's thumb. **IV** Re-fasten the Velcro section.

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